



KONICA MINOLTA

June 10, 2020
Healthcare Business Headquarters
KONICA MINOLTA, Inc

COVID-19 Support from KONICA MINOLTA

Disinfecting for DIRECT DIGITIZER REGIUS SYSTEM

■ Product

REGIUS Σ

REGIUS Σ2, Cassette

REGIUS MODEL 110, Cassette

REGIUS MODEL 210, Cassette

■ Available Disinfectants and Procedure

<Disinfection for Cassette>

- Use ethanol for disinfection or isopropanol.
- To prevent the leakage of the disinfectants into the inside of the cassette, put a small amount of the disinfectants on non-woven fabric and wipe off the cassette surface.
- Disinfection of cassette must be done before it is loaded into reader.

<Disinfection for Reader>

- Use ethanol for disinfection or isopropanol.
- Put the disinfectants on the soft and non-woven fabric, squeeze it well and wipe off the reader surface.
- If the reader is REGIUS MODEL 210, don't use the volatile liquids such as alcohol or thinner to clean the ejection rubber roller because it may cause the cassette jamming due to deteriorating the rubber. (REGIUS MODEL 210 Operation Manual 6.2.3).

■ Caution

- Turn off the power and disconnect the power cable from power source before performing disinfection.
- Using those disinfectants does not guarantee cleaning or disinfection efficacy and have not been tested by KONICA MINOLTA, Inc. for use with the COVID-19 virus. Please follow the policies and procedures of your institution for the cleaning and disinfection of equipment.
- To use those disinfectants, make sure to follow the instructions provided by each manufacturer.