

Caring for your Ultrasound Systems & Transducers: SONIMAGE® HS1 • SONIMAGE® MX1

INSTRUCTIONS FOR DAILY CARE

- Ensure transducer cords are secured to cart. Serious damage will occur if cord is rolled over and/or pulled from under wheels
- **DO NOT** use ethyl chloride cold spray near the transducer. It will cause serious damage, even if a probe cover is in place
- **NOTE:** Betadine and skin markers will cause discoloration of the transducer and cord

Subsequent damage caused by any of the above is **NOT COVERED UNDER WARRANTY**

- Cleanse transducer/console thoroughly after every use with one of the following approved cleaning products:
 - ▶ Protex Disinfectant Wipes or Spray
 - ▶ Super Sani-Cloth (purple top)
 - ▶ Sani-Cloth HB (green top)
 - ▶ Sani-Cloth Plus (red top)
 - ▶ PI Spray
 - ▶ Isopropyl Alcohol up to 70% (do not use alcohol stronger than 70%)
- Use microfiber cloth or cleansing wipe designed for use on medical touchscreens to clean touchscreen
- Touchscreen must be deactivated or system **SHUTDOWN** before cleaning touchscreen

CHARGING/BATTERY LIFE

- System should be plugged in whenever possible. Help preserve battery life by using **STANDBY** mode between patients or by pressing **FREEZE** when system is on and not in use
- Please **SHUTDOWN** system at the end of every day and plug in to recharge

REFERENCE MANUALS

The Cleaning Quick Reference Guide is not a replacement for review of the cleaning procedures stated in the system User Manuals.

HS1 System Operations Manual: Fundamentals

- ▶ Section 8.1, pgs. 240-241

MX1 System Operations Manual: Maintenance & Inspection

- ▶ Sections 9.2.2 and 9.2.3, pgs. 146-148

